

INSTRUCTIONS FOR USE

CTG Belts/Straps



READ INSTRUCTIONS FOR USE



REVIEW CAUTIONS



FOR SINGLE USE ONLY



NON-STERILE



MEDICAL DEVICE



MANUFACTURER



Before using product, read through the following information thoroughly

CAUTION: Federal law restricts this device to sale and use by, or on the order of, a physician.



SUNGO Europe B.V.
Olympisch Stadion 24, 1076DE
Amsterdam, Netherlands

LEGAL MANUFACTURER:

Shenzhen Caremed Medical Technology Co., Ltd.
Production license number:20132481
East side, 3rd Floor, C building, Kelunte
Low-carbon Industrial Park,
Dalang, Longhua, Shenzhen,
China 518109

Tel: 0755-28101223
Fax: 0755-27186486
Web: <http://www.szcaremed.com>
Email: CM001@szcaremed.com

NISSHA
MEDICAL TECHNOLOGIES

HS.NisshaMedical.com

MANUFACTURED FOR:
Nissha Medical Technologies
Torbay Business Park
Woodview Road
Paignton, Devon TQ4 7HP
United Kingdom
HS.NisshaMedical.com

NMT090FTL-UK
Disposable Foetal Belt IFU (en) 4009043C
Rev Date: Sep-21

EN ENGLISH

CTG Belts/Straps For Use with PN 4009043C

INTENDED USE

- The foetal belt product allows the fetal transducer to be fixed and adjusted into place.

PRODUCT INTRODUCTION

- Product name: Disposable Fetal Belt
- Product Contents: 2x disposable elastic foetal belts and buckles

PRODUCT RANGE

- Used to fix the fetal transducer when pregnant in order to monitor the fetal heartbeat.

USE CONDITION

- Temperature: 0°C~40°C
- Relative humidity: 15%~95%
- Atmospheric pressure: 70kpa~106kpa

PACKAGE AND STORAGE

- Disposable fetal belt should be single packaged. Packaged product should be stored in a non-corrosive gas, good ventilation room.
- Store environment as below:
 - Temperature: -10°C~+40°C
 - Relative humidity: ≤80%
 - Atmospheric pressure: 70kpa~106kpa

INSTALLATION/USE

- Attach one end of the belt to the transducer by engaging one of the holes in the belt over the button on the top of the transducer.
- Keeping the transducer in position, tension the other end of the belt and engage the belt over the button, ensuring sufficient tension to keep the transducer in firm contact with the abdomen.
- Avoid over-tightening as this will cause unnecessary discomfort to the patient.

WARRANTY/VALIDITY PERIOD

- Warranty period: N/A
- Validity period: 2 years

REPLACE AND ABANDON

- Replace: Under these situations such as fracture, elastic damage, expiry date, please change new fetal belt to use.
- Abandon: Abandoned belt can be processed as normal waste.

CONTRAINDICATIONS, PRECAUTIONS, WARNINGS AND CAUTIONARY STATEMENTS

- Contraindications: The belts must not be fixed at the injury site.
- Spandex allergies to stop using
- Cautionary statements: This product is a single-use product to avoid cross infection. Please read the instructions before use.