

## CTG BELTS / STRAPS – INSTRUCTIONS FOR USE

### Intended use:

- This product works according to adjust buckle's place on the belt to fix the fetal transducer.

### Product introduction :

- Product name: Disposable fetal belt
- Product structure: It's made up of elastic belt and buckle.

### Product range :

- It used with fetal transducer when pregnant in fetal monitoring, to fix the fetal transducer.

### Use condition :


- Temperature:0°C~40°C ;
- Relative humidity:15%~95% ;
- Atmospheric pressure:70kpa~106kpa

### Package and storage :

- Disposable fetal belt should be single packaged, packaged product should be stored in a non-corrosive gas, good ventilation room.
- Store environment as below:
  - 1, Temperature:-10°C~+40°C ;
  - 2, Relative humidity:≤80% ;
  - 3, Atmospheric pressure:70kpa~106kpa.

### Installation/use :

- Attach one end of the belt to the transducer by engaging one of the holes in the belt over the button on the top of the transducer.
- Keeping the transducer in position, tension the other end of the belt and engage the belt over the button, ensuring sufficient tension to keep the transducer in firm contact with the abdomen.
- Avoid over-tightening as this will cause unnecessary discomfort to the patient.

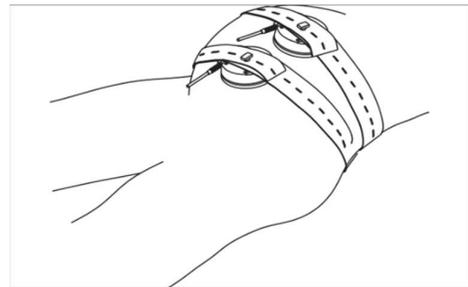

### Product clean :

- Wash belt in warm water, do not rub the edge fetal belt to avoid removal, do not exposure.

### Warranty/validity period :

- Warranty period: N/A.
- Validity period:2 years.


### Replace and abandon:

- Replace: Under these situations such as fracture, elastic damage, expiry date, please change new fetal belt to use.
- Abandon: Abandoned belt can be processed as normal waste.

### Contraindications, precautions, warnings and cautionary statements:

- Contraindications: The belts must not be fixed at the injury site.
- Spandex allergies to stop using.
- Cautionary statements: This product is a single-use product to avoid cross infection. Please read the instructions before use.

### Mark :

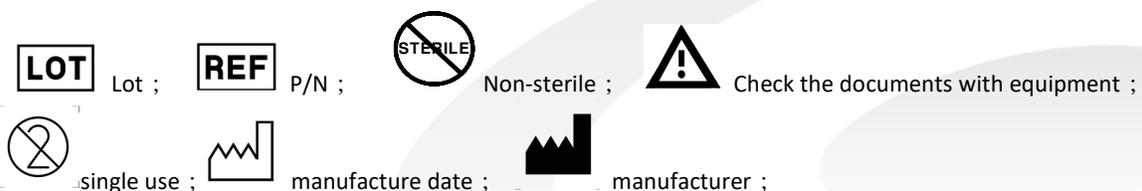

Manufacturer/Registrant: Shenzhen Caremed Medical Technology Co., Ltd.

Production license number: 粤食药监械生产许 20132481 号

Address:3rd floor,11 building, Hebei Industrial Area, Longhua, Shenzhen City, China 518100 Registration

address:3rd floor,11 building, Hebei Industrial Area, Longhua, Shenzhen City, China 518100 Tel:0755-

27184369 Fax:0755-27186486 Web: <http://www.szcaremed.com> Email: [CM001@szcaremed.com](mailto:CM001@szcaremed.com)